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Step 1: Please assemble the reagent rack as shown in the pictures! Note: The test device and reagents must be at room temperature for at least 30 min before use.

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Step 3a (washing): Pipette the entire volume of the washing solution (WASH) (blue cap) into the device. **Step 3b:** Pipette the entire volume of the antibody solution 1 (Ab1) (grey cap) into the device. **Inkubation: 8 Min**

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
Step 5a (washing): Pipette the entire volume of the washing solution (WASH) (blue cap) into the device. **Step 5b:** Pipette the entire volume of colour reagent (COLR) from the first vial (white cap) into the device, start the timer, and then immediately pipette the colour reagent (COLR) from the second vial into the device. **Total Incubation: 5 Min**

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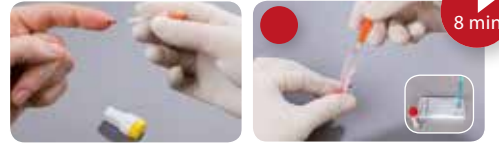Step 2a: Collect 200 µL of blood (up to the white barrier) with the blood-sampling capillary (67 µL of serum or plasma can also be used) **Step 2b:** Transfer the blood sample into the sample dilutor (DILUT) (red cap) and mix for two seconds. **Step 2c:** Pipette (using the syringe) the entire volume of the blood sample mix into the fluid port of the device. **Inkubation: 8 Min**

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Step 4a (washing): Pipette the entire volume of the washing solution (WASH) (blue cap) into the device. **Step 4b:** Pipette the entire volume of the antibody solution 2 (Ab2) (yellow cap) into the device. **Inkubation: 8**

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Step 6: Pipette the entire volume of the stop buffer (STOP) (green cap) into the device

Done! Check the test results using the evaluation sheet!

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