

# Quo-Test How to run a test



## Step 1

Ensure that the analyser is displaying the home screen.



## Step 2

Scan appropriate lot bar code if required.




## Step 3

Remove cartridge and capillary from the foil package.



## Step 4

Finger prick patient.

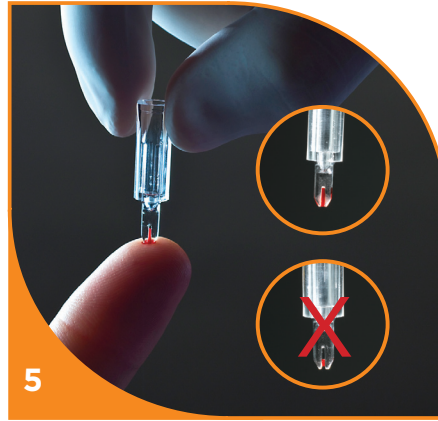

## Step 5

Take blood sample ensuring capillary is filled correctly.



## Step 6

Insert capillary into Quo-Test cartridge.



## Step 7

Insert cartridge into the Quo-Test.



## Step 8

Lock cartridge in position by pulling the red tab forward and close the instrument door.



## Step 9

Results are displayed on the screen within 4 minutes.